

# CAMTEC BED SCALE INSTRUCTIONS

## Scale Instructions:

Apply power to bed.

Press & release **Freeze / Resume**

Wait 5 seconds and CAMTEC will be replaced by "weight XXXX lbs."

### Now Two Choices

#### Choice One - Patient Not in Bed

Step One - Place all inventory to be used by patient on the bed such as pillows, blankets, etc.

Step Two - Press **0.0** hold and follow instructions.

Step Three - Put patient in bed.

Step Four - Press **WEIGHT MODE** momentarily - read patient weight.

THE END

#### Choice Two - Patient in bed and patient weight known

Step One - Press **FREEZE / RESUME** and follow instructions on screen.

Step Two - When screen reads FREEZE change weight reading up or down by pressing and holding **WEIGHT MODE** (to increase) or **WT / CHANGE MODE** (to decrease) until correct patients weight appears on screen. If screen changes to "LEFT IN FREEZE / PRESS FREEZE" press and release **FREEZE / RESUME**. Then resume weight adjustment.

Step Three - Press **FREEZE / RESUME** and follow instructions.

Step Four - Press **WEIGHT MODE** momentarily - read patient weight.

THE END

903492-0