


# NOA Low Bed

## Quick Reference Guide

- Caster brakes should be kept locked when in use.
- Consider the use of floor pads to increase patient safety.

### BED OPERATION

Bed Positioning – Bed functions are operated with either an 8 or 10 button pendant. A 10 button pendant with lock out feature is shown below. Button features on the pendant are clearly identified with universal icons.

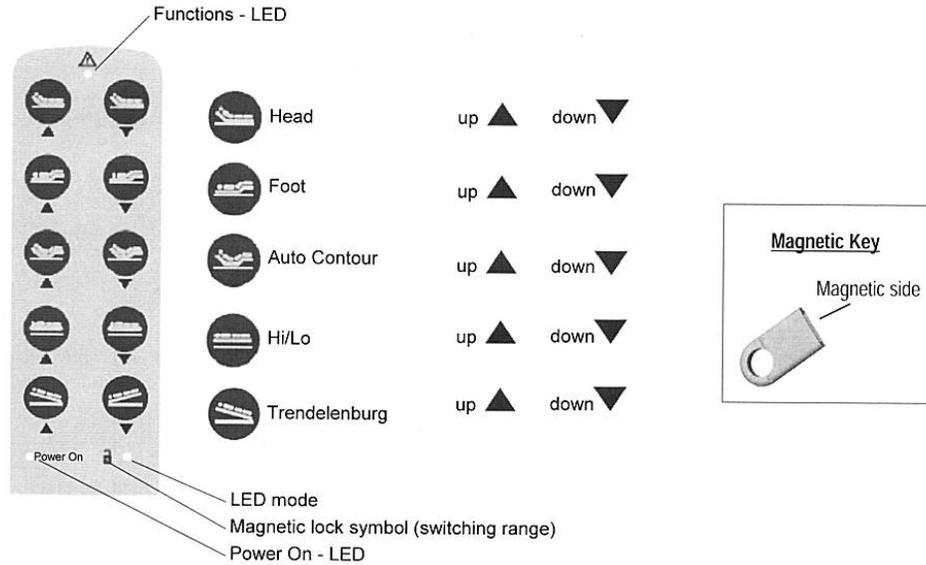

### Operation of the Lock Out Pendant

By passing the magnetic side of the magnetic key over the lock symbol, three modes can be released or locked.

- To put the Lock Out Pendant into operation, connect it to the MCL Control Box through the "Y" cable. The keyboard of the pendant is completely blocked (mode 1). The LED next to the lock symbol **does not glow**.
- Passing the magnetic side of the magnetic key once over the lock symbol releases the resident comfort functions (mode 2). The pendant will stay in this mode until the magnetic key is once again passed over the lock symbol. The LED next to the lock symbol **glows green**.
- Passing the magnetic side of the magnetic key once again over the lock symbol releases the HI/LO and Trendelenburg/Reverse Trendelenburg functions (mode 3). The LED next to the lock symbol **glows amber**.

While in mode 3 if no button is pressed after approximately 50 seconds of the pendant having last been actuated, mode 1 is reset.

<b>NOA Specifications</b>			
Overall Length	84"	Inside Length Head to Footboard	82"
Overall Width (with rails)	40"	Deck Height (High)	26"
Overall Width (w/out rails)	35"	Deck Height (Low)	7"
Deck Width	35"	Maximum Head Elevation	70 deg
Deck Surface Length	80"	Maximum Hip to Knee Angle	45 deg
Weight Capacity	500 lb	Deck to Top of SideRail	17.75"